


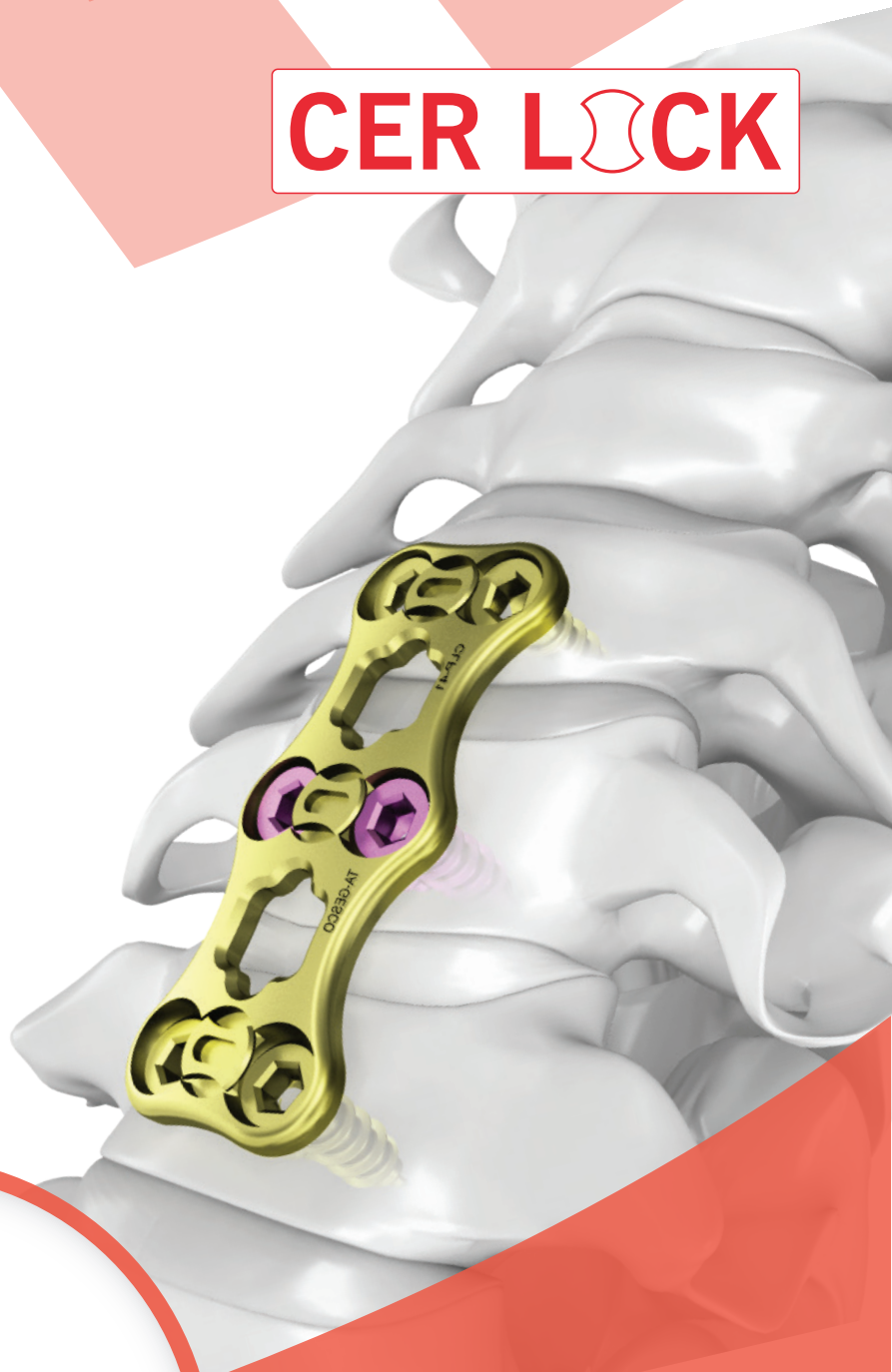

CER LOCK



Alleviating Pain
SINCE 1962

Anterior Cervical Plate System



Step 1 : The plate length should be defined according to the chosen screw length and angulation so that it does not interfere with the adjacent unfused disc spaces.

Step 2 : The Plate Bender may be utilized to optimally contour the sagittal plane to ensure maximum bone/plate interface.




Step 3 : Fixation Pins may be used as a means of temporary fixation during subsequent steps.



Step 4 : After confirming proper plate positioning, align the cut-out on the distal tip of the Drill Guide over the Securement Tab on the plate. Press down on the proximal end of the drilling guide while inserting its distal tip into the plate's screw hole. The tap may now be used to thread the holes.



Step 5 : Use the Cervical Friction Spring Driver, pick up the screw from the tray and verify the screw length using the length gauge built into the screw caddy and advance the screws in to the body.

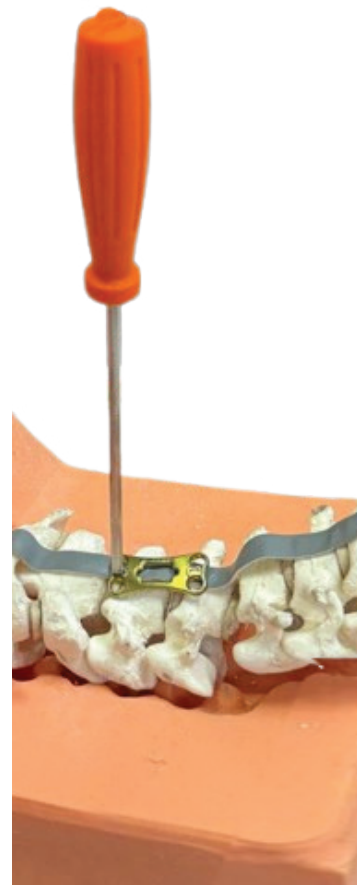

Step 6 : Perform final tightening of all screws using the Cervical Screw Driver. VARIABLE.

The image displays a variety of dental implants and surgical instruments. The top section features a 'BOTTOM TRAY' containing several long, thin screws with blue handles, labeled 'CLI-TFP TEMPORARY FIXATION PIN' and 'CLI-TFP TEMPORARY FIXATION PIN HANDLE'. Below this is a 'CLI-SHS SCREW HOLDING SLEEVE' and a 'PIN HANDLE'. The bottom section shows a 'CERLOCK IMPLANTS & INSTRUMENTATION' tray with a grid of screws and a row of gold-colored pins. The screws are labeled with sizes: 4.0x15, 4.0x20, 4.0x25, 4.0x30, 4.0x35, 4.0x40, 4.0x45, 4.0x50, 4.0x55, 4.0x60, 4.0x65, 4.0x70, 4.0x75, 4.0x80, 4.0x85, 4.0x90, 4.0x95, 4.0x100, 4.0x105, 4.0x110, 4.0x115, 4.0x120, 4.0x125, 4.0x130, 4.0x135, 4.0x140, 4.0x145, 4.0x150, 4.0x155, 4.0x160, 4.0x165, 4.0x170, 4.0x175, 4.0x180, 4.0x185, 4.0x190, 4.0x195, 4.0x200, 4.0x205, 4.0x210, 4.0x215, 4.0x220, 4.0x225, 4.0x230, 4.0x235, 4.0x240, 4.0x245, 4.0x250, 4.0x255, 4.0x260, 4.0x265, 4.0x270, 4.0x275, 4.0x280, 4.0x285, 4.0x290, 4.0x295, 4.0x300, 4.0x305, 4.0x310, 4.0x315, 4.0x320, 4.0x325, 4.0x330, 4.0x335, 4.0x340, 4.0x345, 4.0x350, 4.0x355, 4.0x360, 4.0x365, 4.0x370, 4.0x375, 4.0x380, 4.0x385, 4.0x390, 4.0x395, 4.0x400, 4.0x405, 4.0x410, 4.0x415, 4.0x420, 4.0x425, 4.0x430, 4.0x435, 4.0x440, 4.0x445, 4.0x450, 4.0x455, 4.0x460, 4.0x465, 4.0x470, 4.0x475, 4.0x480, 4.0x485, 4.0x490, 4.0x495, 4.0x500, 4.0x505, 4.0x510, 4.0x515, 4.0x520, 4.0x525, 4.0x530, 4.0x535, 4.0x540, 4.0x545, 4.0x550, 4.0x555, 4.0x560, 4.0x565, 4.0x570, 4.0x575, 4.0x580, 4.0x585, 4.0x590, 4.0x595, 4.0x600, 4.0x605, 4.0x610, 4.0x615, 4.0x620, 4.0x625, 4.0x630, 4.0x635, 4.0x640, 4.0x645, 4.0x650, 4.0x655, 4.0x660, 4.0x665, 4.0x670, 4.0x675, 4.0x680, 4.0x685, 4.0x690, 4.0x695, 4.0x700, 4.0x705, 4.0x710, 4.0x715, 4.0x720, 4.0x725, 4.0x730, 4.0x735, 4.0x740, 4.0x745, 4.0x750, 4.0x755, 4.0x760, 4.0x765, 4.0x770, 4.0x775, 4.0x780, 4.0x785, 4.0x790, 4.0x795, 4.0x800, 4.0x805, 4.0x810, 4.0x815, 4.0x820, 4.0x825, 4.0x830, 4.0x835, 4.0x840, 4.0x845, 4.0x850, 4.0x855, 4.0x860, 4.0x865, 4.0x870, 4.0x875, 4.0x880, 4.0x885, 4.0x890, 4.0x895, 4.0x900, 4.0x905, 4.0x910, 4.0x915, 4.0x920, 4.0x925, 4.0x930, 4.0x935, 4.0x940, 4.0x945, 4.0x950, 4.0x955, 4.0x960, 4.0x965, 4.0x970, 4.0x975, 4.0x980, 4.0x985, 4.0x990, 4.0x995, 4.0x1000, 4.0x1005, 4.0x1010, 4.0x1015, 4.0x1020, 4.0x1025, 4.0x1030, 4.0x1035, 4.0x1040, 4.0x1045, 4.0x1050, 4.0x1055, 4.0x1060, 4.0x1065, 4.0x1070, 4.0x1075, 4.0x1080, 4.0x1085, 4.0x1090, 4.0x1095, 4.0x1100, 4.0x1105, 4.0x1110, 4.0x1115, 4.0x1120, 4.0x1125, 4.0x1130, 4.0x1135, 4.0x1140, 4.0x1145, 4.0x1150, 4.0x1155, 4.0x1160, 4.0x1165, 4.0x1170, 4.0x1175, 4.0x1180, 4.0x1185, 4.0x1190, 4.0x1195, 4.0x1200, 4.0x1205, 4.0x1210, 4.0x1215, 4.0x1220, 4.0x1225, 4.0x1230, 4.0x1235, 4.0x1240, 4.0x1245, 4.0x1250, 4.0x1255, 4.0x1260, 4.0x1265, 4.0x1270, 4.0x1275, 4.0x1280, 4.0x1285, 4.0x1290, 4.0x1295, 4.0x1300, 4.0x1305, 4.0x1310, 4.0x1315, 4.0x1320, 4.0x1325, 4.0x1330, 4.0x1335, 4.0x1340, 4.0x1345, 4.0x1350, 4.0x1355, 4.0x1360, 4.0x1365, 4.0x1370, 4.0x1375, 4.0x1380, 4.0x1385, 4.0x1390, 4.0x1395, 4.0x1400, 4.0x1405, 4.0x1410, 4.0x1415, 4.0x1420, 4.0x1425, 4.0x1430, 4.0x1435, 4.0x1440, 4.0x1445, 4.0x1450, 4.0x1455, 4.0x1460, 4.0x1465, 4.0x1470, 4.0x1475, 4.0x1480, 4.0x1485, 4.0x1490, 4.0x1495, 4.0x1500, 4.0x1505, 4.0x1510, 4.0x1515, 4.0x1520, 4.0x1525, 4.0x1530, 4.0x1535, 4.0x1540, 4.0x1545, 4.0x1550, 4.0x1555, 4.0x1560, 4.0x1565, 4.0x1570, 4.0x1575, 4.0x1580, 4.0x1585, 4.0x1590, 4.0x1595, 4.0x1600, 4.0x1605, 4.0x1610, 4.0x1615, 4.0x1620, 4.0x1625, 4.0x1630, 4.0x1635, 4.0x1640, 4.0x1645, 4.0x1650, 4.0x1655, 4.0x1660, 4.0x1665, 4.0x1670, 4.0x1675, 4.0x1680, 4.0x1685, 4.0x1690, 4.0x1695, 4.0x1700, 4.0x1705, 4.0x1710, 4.0x1715, 4.0x1720, 4.0x1725, 4.0x1730, 4.0x1735, 4.0x1740, 4.0x1745, 4.0x1750, 4.0x1755, 4.0x1760, 4.0x1765, 4.0x1770, 4.0x1775, 4.0x1780, 4.0x1785, 4.0x1790, 4.0x1795, 4.0x1800, 4.0x1805, 4.0x1810, 4.0x1815, 4.0x1820, 4.0x1825, 4.0x1830, 4.0x1835, 4.0x1840, 4.0x1845, 4.0x1850, 4.0x1855, 4.0x1860, 4.0x1865, 4.0x1870, 4.0x1875, 4.0x1880, 4.0x1885, 4.0x1890, 4.0x1895, 4.0x1900, 4.0x1905, 4.0x1910, 4.0x1915, 4.0x1920, 4.0x1925, 4.0x1930, 4.0x1935, 4.0x1940, 4.0x1945, 4.0x1950, 4.0x1955, 4.0x1960, 4.0x1965, 4.0x1970, 4.0x1975, 4.0x1980, 4.0x1985, 4.0x1990, 4.0x1995, 4.0x2000, 4.0x2005, 4.0x2010, 4.0x2015, 4.0x2020, 4.0x2025, 4.0x2030, 4.0x2035, 4.0x2040, 4.0x2045, 4.0x2050, 4.0x2055, 4.0x2060, 4.0x2065, 4.0x2070, 4.0x2075, 4.0x2080, 4.0x208

TOP TRAY

CLI-PD PIN DISTRACTOR

CLI-P14 PIN 1.4 mm

CLI-PB PIN

CLI-OS OVAL SCREW DRIVER

CERCLON PLATE HOLDER

CERCLON IMPLANTS & INSTRUMENTATION

